

## AIO 3.0 - SBRT PRESSURE SYSTEM BRIDGE LOW – HEXAPOD™ COMPATIBLE

Article Nos. : **38240**



### A. GENERAL PRODUCT INFORMATION

The product referred to in these instructions is a medical device, used for patient positioning and immobilisation in radiation therapy. This product can be used during both the simulation and the treatment stage, including MRI simulation.

To attain an optimal result, it is recommended to use this product in combination with Orfit immobilisation products.

### B. PRODUCT DESCRIPTION

This arch is used as a compression device for the abdomen. It clamps onto the AIO 3.0 SBRT compatible and HexaPOD™ compatible base plates ref. 38019, 38020 & 38021. The compression plate is screwed downwards to reach the desired pressure on the abdomen and as such immobilise the abdomen. Information on these other parts (and instructions on how to make the masks) can be found in the respective 'instructions for use' and on [www.orfit.com](http://www.orfit.com).

### C. PRECAUTIONS FOR USE

- This is a fragile product, please handle with care.
- Make sure the device is securely locked onto the AIO 3.0 SBRT compatible HexaPOD™ compatible base plates ref. 38019, 38020 & 38021 on both sides.
- The pressure system bridge can only be used with the AIO 3.0 SBRT compatible and HexaPOD™ compatible base plates ref. 38019, 38020 & 38021. This bridge is not compatible with the other AIO 3.0 SBRT base plates.

### D. INSTALLATION

1. Position the arch in the highest position by opening the locks (A) on both sides and moving the arch vertically upwards to position 5. Close the locks again. (Figure 1)

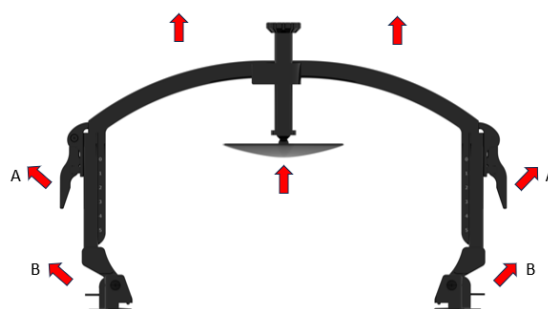


Figure 1

2. Position the screw in the highest position by opening the locking system, pulling the screw upwards and closing the locking system again. (Figure 2)



Figure 2

3. Place the pressure plate (32317/2/12) in the correct position on the abdomen of the patient. Be aware that the plate does not exercise pressure on the xiphoid process or ribs to prevent fractures. (Figure 3)



Figure 3

- Position the arch on the baseplate so the screw will touch the pressure plate once you screw it down. Attach the arch to the base plate by opening the clamps (B) on both sides. The arch indexes onto the round holes on top of the base plate and clamps underneath the thin edge. For repositioning purposes write down the location of the arch. The different locations are numbered on the base plate E1 to E26. (Figure 4)

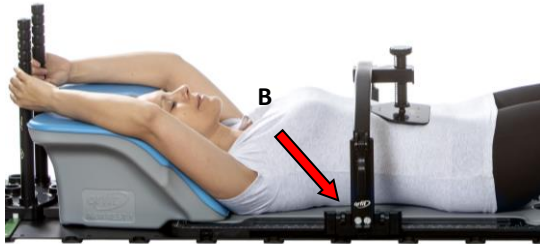


Figure 4

- !! Note that the arch can be positioned with the screw facing the caudal end of the base plate and also the other way around with the screw facing the cranial end of the baseplate. Choose the correct position so the arch is located outside of the treatment field.
- Move the arch vertically downwards until the screw touches the pressure plate by opening the locks (A) on both sides. When you have reached the correct position, close the locks again. When releasing the lock handles make sure to hold the frame to prevent it from sliding down on the patient.

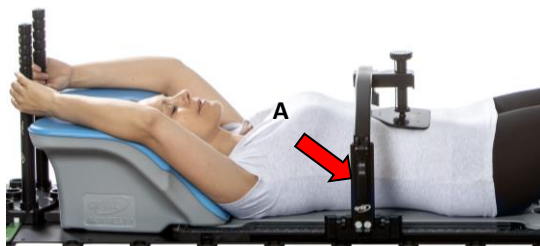


Figure 5

- Fasten the screw until you have reached the desired pressure or height on the abdomen. If the standard screw doesn't reach deep enough, switch to the longer screw.



Figure 6

- The height of the screw can be read from the ruler (Figure 7) or measured for future reference.



Figure 7

- Make sure to note all indexing positions of the arch prior to release.

In case of an emergency, pressure can be released quickly:

- by opening the vertical screw locking handle and pulling the screw upwards.

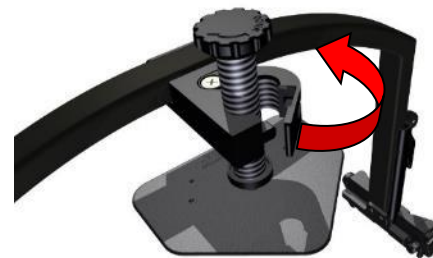


Figure 8

- by releasing the arch to base plate handles and pulling the arch backwards.



Figure 9

**Minimum and maximum height (with standard screw):**

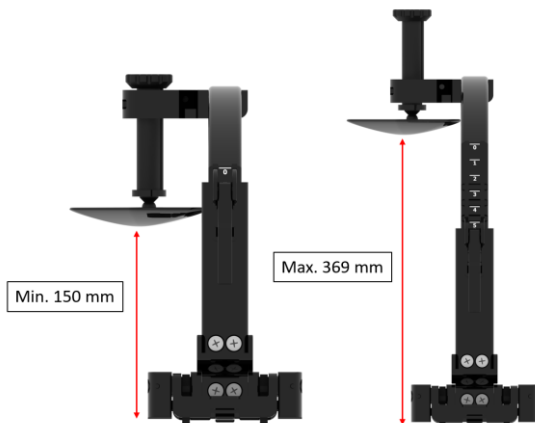


Figure 10

## E. STORAGE

Always store the product in a safe place to prevent it from getting damaged.

Store the arch between +10°C (50°F) and 40°C (122°F).

## F. PROPERTIES

### F.1. Physical Properties 38240

Dimensions: W 195mm x H 380mm x L 630 mm

Weight: 2.35 kg

Materials: HPL, POM, PEEK, PA

Average density: 1,4 gr/cm<sup>3</sup>

The arch is made entirely of electrically non-conductive, non-metallic and non-magnetic materials and is MRI safe.



## F.2. Mechanical Properties

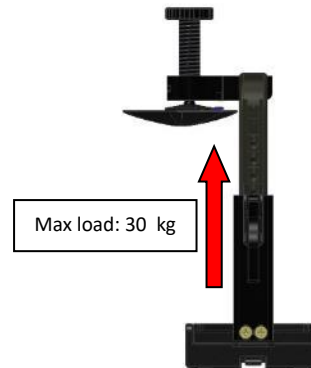


Figure 11

Component failure can occur when max load is exceeded typically because the patient exerts high forces on the device (panic, coughing, vomiting, etc.). Component failure is not harmful for the patient.

## G. MAINTENANCE AND WASTE MANAGEMENT

This product can be cleaned and disinfected by means of an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. **Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents.**

Periodic checks of the product should be done to insure the parts are not worn and require repair or replacement. **Do not attempt to make repairs yourself.** Do not use the product when damaged. Contact your distributor if there are any questions or concerns.

The product can be disposed of with household waste.

## H. ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com).

### Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries. RAYCAST™ and EFFICAST™ are registered trademarks of ORFIT Industries. HexaPOD™ is a trademark of the Elekta Group.



**ORFIT INDUSTRIES**  
Vosveld 9A | B-2110 Wijnegem | Belgium  
T (+32) (0)3 326 20 26  
welcome@orfit.com

[www.orfit.com](http://www.orfit.com)



Ref. No. 50295  
VERSION 1  
LAST UPDATE: 30/11/2023  
REVISION DATE: 30/11/2025